INSTRUCTIONS FOR

Vira x2 - BINDER

art no 50228

Application

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EN Choose size and place the binder over the surged area. Begin by fastening the lower band **1** at the bottom of the hip. Then fasten the top band **2**. Adjust until it feels comfortable. The girdle should sit tight but not uncomfortably tight.

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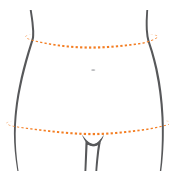
EN Choose the size by the hip circumference, if the hip size is within an overlapping range, check the circumference of your waist:

- Narrow waistline: choose the smaller size.
- Straight body shape: choose the larger size.

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EN The initial fitting, prescribing and recommendations for use should be done by/under the direction of qualified medical practitioner or other professionals. Follow the instructions for best result. If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person.

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EN For postoperative care after abdominal surgery, abdominoplasty and/or liposuction. Can also be used for various types of hernia problems. Relieves wounds, provides pain relief, and enables early mobilization of the patient.

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EN Periodically check the condition of the velcro to ensure they are free from dirt and that their fastening capabilities have not deteriorated. A qualified medical practitioner should supervise use in combination with other medical products.

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EN Machine wash at 40°C. Can be washed at a higher temperature if necessary, max. 60°C. The product will last longer if washed at a lower temperature. Use a laundry bag. Dry flat. Do not tumble dry. Seal the velcros before washing. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material.

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